Nederlands

De Conveen Contour Beenzak is beschikbaar in 2 maten: 600 ml (ref. 5170) en 800 ml (5176-NL/ref. 5174-BE)

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www.coloplast.com

23702120

23702120 Coloplast and Conveen are registered trademarks of Coloplast A/S © 05/04. All rights reserved Coloplast A/S, DK-3050 Humlebæk, Denmark Manufacturer: Coloplast A/S, DK-3050 Humlebæk, Denmark

0543

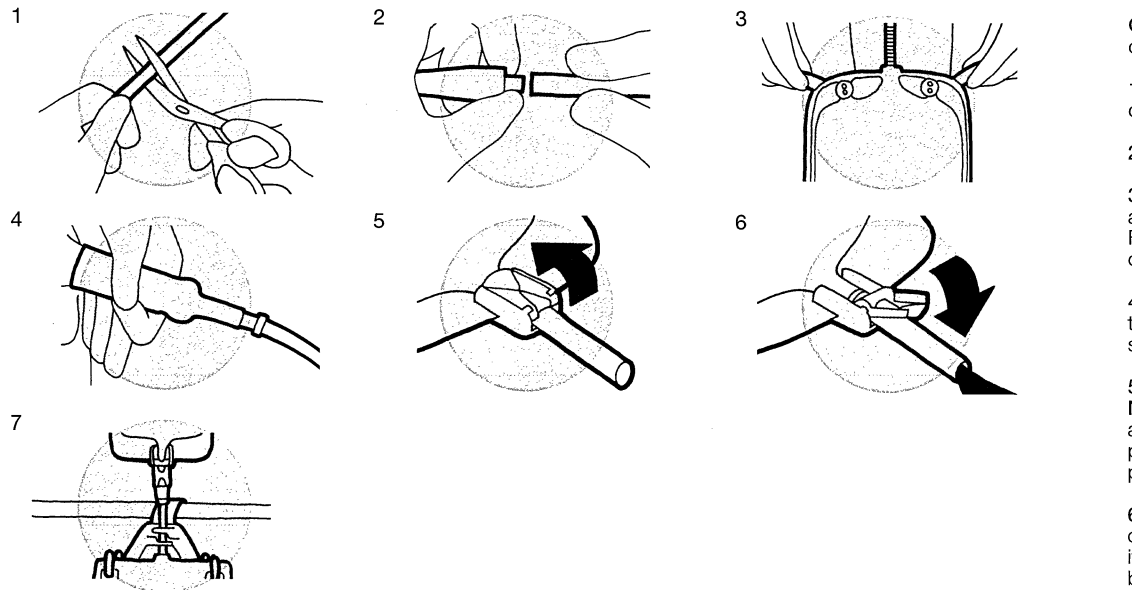
Contour leg bag
Mehrkammer-Beinbeutel
Poche à urine Contour
Bolsa de orina Multicanal
Sacca da Gamba Contoured
Contour Beenzak
Flerkammarpåse
Muotoiltu Contour-virtsapussi
Flerkammerbenpose

Conveen

Illustration

English

Conveen Contour Leg Bag has a 600 ml or 800 ml capacity with adjustable inlet tube.

1. The tube may be shortened to desired length using ordinary scissors.

2. Attach the connector to the tube.

3. Fasten the bag to your leg by using the straps provided and fastening the buttons through the slits at each end. Fit the bag securely to your leg, adjusting the straps for a comfortable fit. Any surplus strap can be trimmed off.

4. Attach the connector securely to the sheath and ensure that neither the urisheath nor the tubing is bent or kinked so as not to obstruct the flow of urine.

5. Ensure the outlet tap is in the closed position.
NB The outlet tap of the Conveen Contour Leg Bag is always open when the bags are packed. To close the tap push the clamp lever upwards until you feel it click into place. Always remember to close the tap after emptying.

6. To empty the bag, direct the outlet tube into the toilet or other suitable receptacle and open the tap. You may find it easier if you unbutton or remove the bottom leg strap before emptying.

7. Conveen Contour Leg Bag may easily be connected to the Conveen Night Drainage 1500 ml (Product Code 5062). Ensure that the outlet tap on the night drainage bag is closed. Push the drainage bag connector firmly into the outlet tube of the Conveen Contour Leg Bag; open the leg bag tap. To remove the night drainage bag: firstly close the tap on the leg bag, gently twist the connector of the night drainage bag away from the leg bag outlet tube. Avoid touching the open end of the connector and always wash your hands before and after connecting/disconnecting.

Note: The Drug Tariff (GB), published by the Department of Health, advises that with proper care, leg bags such as this may on average be used for 5-7 days. Disposal of the urine bags should be carried out according to your agreed District Policy (GB).

Sterile products:
Single use only. Do not use if seal is broken. Do not use or re-sterilise product.

Coloplast accepts no liability for any injury or other loss that may arise in the use of this product in a manner contrary to Coloplast's current recommendations.

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